

Vented Needle


REF 415070

STERILE EO Sterilized using ethylene oxide


 Single sterile barrier system

Not made with natural rubber latex.

 Single Use

 Non-pyrogenic

Rx only

 Do not resterilize

 Consult Electronic Instructions for Use
www.bbraunusa.com

CAUTION: NOT FOR DIRECT INFUSION.

Instructions for Use: Use Aseptic Technique.

- for drug reconstitution

1. Using guard as a wrench, attach hub firmly to luer fitting of solution dispensing device.
2. Pull guard straight off - **DO NOT TWIST** - and insert vented needle into rubber stopper of vial. Cleanse site before injection.

3. Inject fluid through vented needle into the vial.
4. Vent slot on needle permits air to escape during injection, preventing a pressure build up inside vial.
5. Remove vented needle before aspirating solution from vial.

A4802219-6 REV. 12/23

B. Braun Medical Inc.
Bethlehem, PA 18018-3524 USA
1-800-227-2862

B | BRAUN