

August 2022

RE: Dialog+ Component Update Notification

Dear Valued B. Braun Customer,

B. Braun Medical Inc. is releasing a new coil for the balance chamber component. This letter is to provide you with a summary of these changes and to ensure a seamless experience with your Dialog machine, as applicable.

The balance chamber coil version 3, part number 34561889, has been discontinued. The replacement for the 34561889 coil version 3 is part number 34562293 coil. The replacement of the old coil (as part of a repair) will require the replacement of the 2 outer parts of the balance chamber (34562292) and 2 new coils will need to be installed. You will also need the repair kit balance chamber (3451897B) as this will have the associated O-rings and hardware to make the changes.

This change is only required to change from the old coil to the new coil as needed for a repair. If your machines already have the new coils or you have made this change as part of a repair already, you can order the individual components as needed for any additional repairs.


34562292	Balance Chamber Outer Part	2 pieces required per machine
34562293	Coil	2 pieces required per machine
3451897B	Repair Kit Balance Chamber	1 piece required per machine

This is not a required upgrade. This is for REPAIRS only!

Please consult with your B. Braun Representative for further information.

Should you have further questions, **please contact our B. Braun Medical Avitum Technical Support Team at 1-800-621-0445 or via JIRA Service Desk.**

Sincerely,

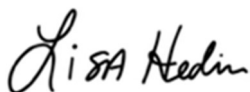

Lisa Hedin
Product Director, Avitum