

APEX® Compounding System Non-Patch Related Security Updates

<p>ISSUE: cifs-generic-0005: Weak LAN Manager hashing permitted (NTLMv1 allowed)</p>	<p>ACTION:</p> <ul style="list-style-type: none"> • Configure the policy value for Computer Configuration -> Windows Settings -> Security Settings -> Local Policies -> Security Options -> "Network security: LAN Manager authentication level" to "Send NTLMv2 response only. Refuse LM & NTLM". • Configure the policy value for Computer Configuration >> Windows Settings >> Security Settings >> Local Policies >> Security Options >> "Network security: Minimum session security for NTLM SSP based (including secure RPC) clients" to "Require NTLMv2 session security" and "Require 128-bit encryption" (all options selected). <p>Adobe Reader at 20.012.20043</p>
<p>Adobe Flash removal</p>	<p>kb4577586 - needed to be pulled from the MS catalog for Windows 1607 x64</p>
<p>Enable the Microsoft patch MS13-098: Vulnerability in Windows Could Allow Remote Code Execution (KB 2893294) as described in CVE-2013-3900 - Security Update Guide - Microsoft - WinVerifyTrust Signature Validation Vulnerability for 64 bit machines.</p>	

Last Updated: 2/21/23