
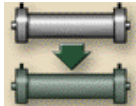



Changing a Diacap Ultra Dialysate Filter

Changing a Diacap Ultra Dialysate Filter



1. Press “**Disinfection**” from the **Program Selection** screen.
2. Press the yellow folder  icon.
3. Press the  icon.

Changing a Diacap Ultra Dialysate Filter

The following message will be displayed:

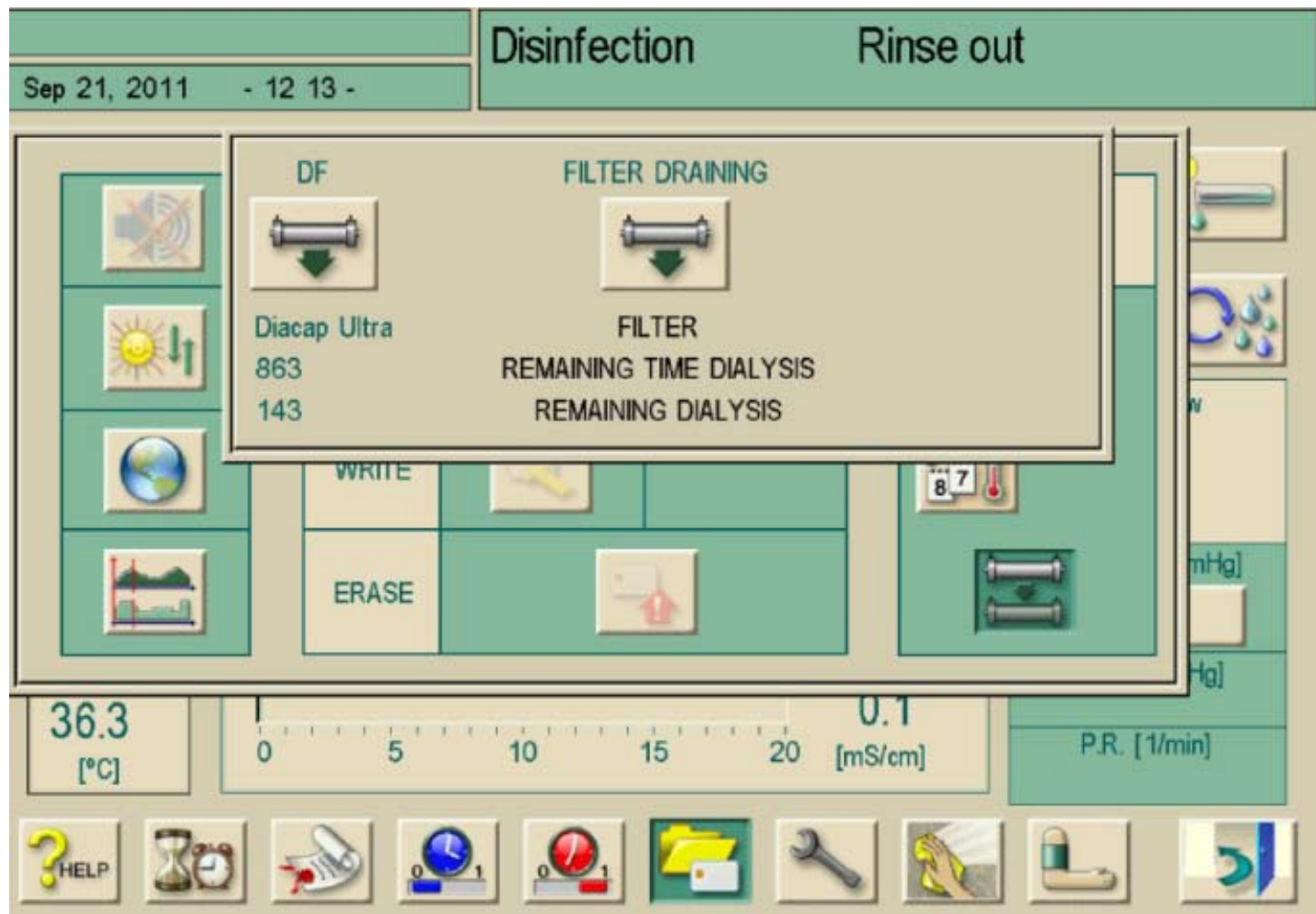


Changing a Diacap Ultra Dialysate Filter

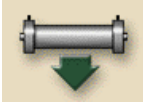


5. Press the  icon
6. Press the  key to answer the question:
“Do you want to drain filter?”
6. Remove BLUE dialyzer coupling from rinse bridge on right side of Dialog machine
7. Wait approximately 90 seconds for the Diacap Ultra filter to drain.

Changing a Diacap Ultra Dialysate Filter

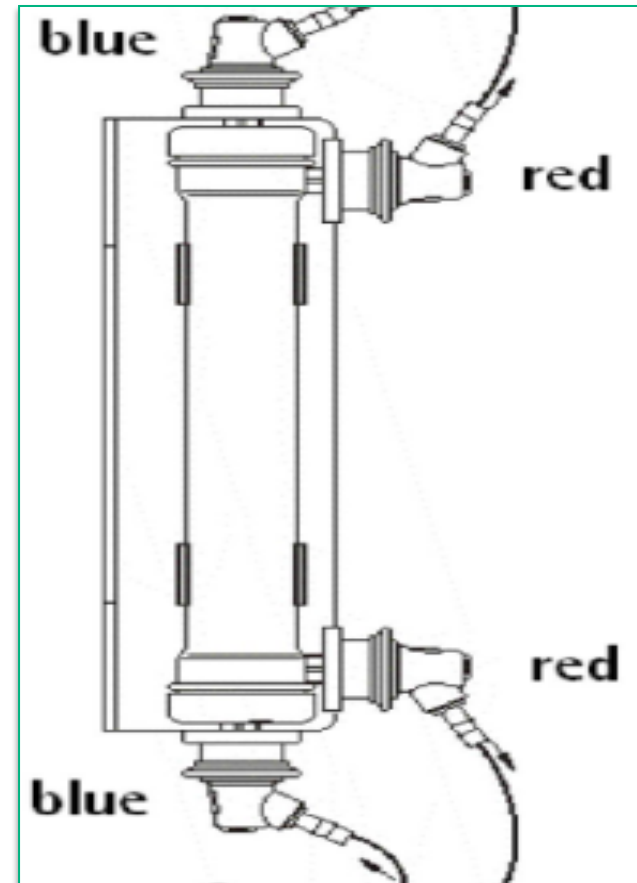8. Remove all connections top to bottom from the filter, noting placement of blue (on filter ends) and red connectors (on filter side).
9. Use plugs from new Diacap Ultra Filter to cap ports on spent Diacap Ultra Filter.
10. Place all couplings on the new Diacap Ultra Filter from bottom to top noting placement of blue (on filter ends) and red connectors (on filter side).

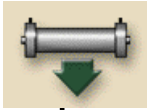

Changing a Diacap Ultra Dialysate Filter

11. Return blue coupling to rinse bridge on the right side of Dialog machine.
12. Tap the  icon. Then confirm with the  key to reset the operating time and number of treatments.
13. Press the  key a second time to get back to the “**Disinfection**” screen
14. Machine **MUST** perform a Citric Thermal DISINFECT prior to patient care.