

Heparin syringe loading technique

January 2016


RTD.TR85 01/2016 JB
©2016 B. Braun Medical Inc.



B | BRAUN
SHARING EXPERTISE

Legend

1. Syringe bracket
2. Syringe gripping plate
3. Clip
4. Unlocking lever
5. Syringe stop

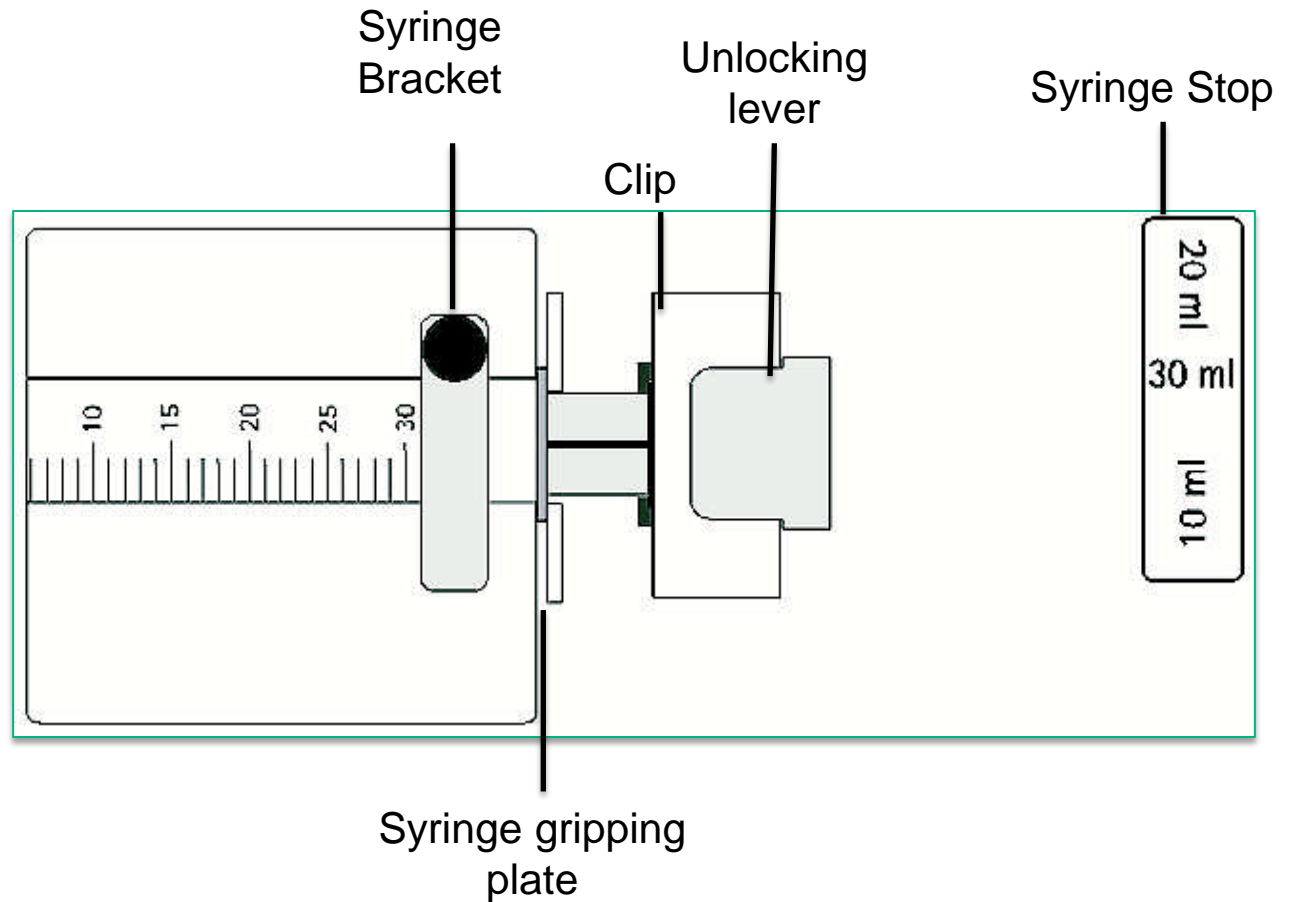

Attaching heparin syringe to heparin administration line

Before attaching the syringe to the heparin administration line, insure that both the syringe and administration line are completely free of air.

Rationale:

Even the smallest amount of air can allow the migration of blood into the syringe.




Attaching heparin syringe to heparin administration line

Aseptically attach the syringe to the Heparin administration line.



Inserting the heparin syringe

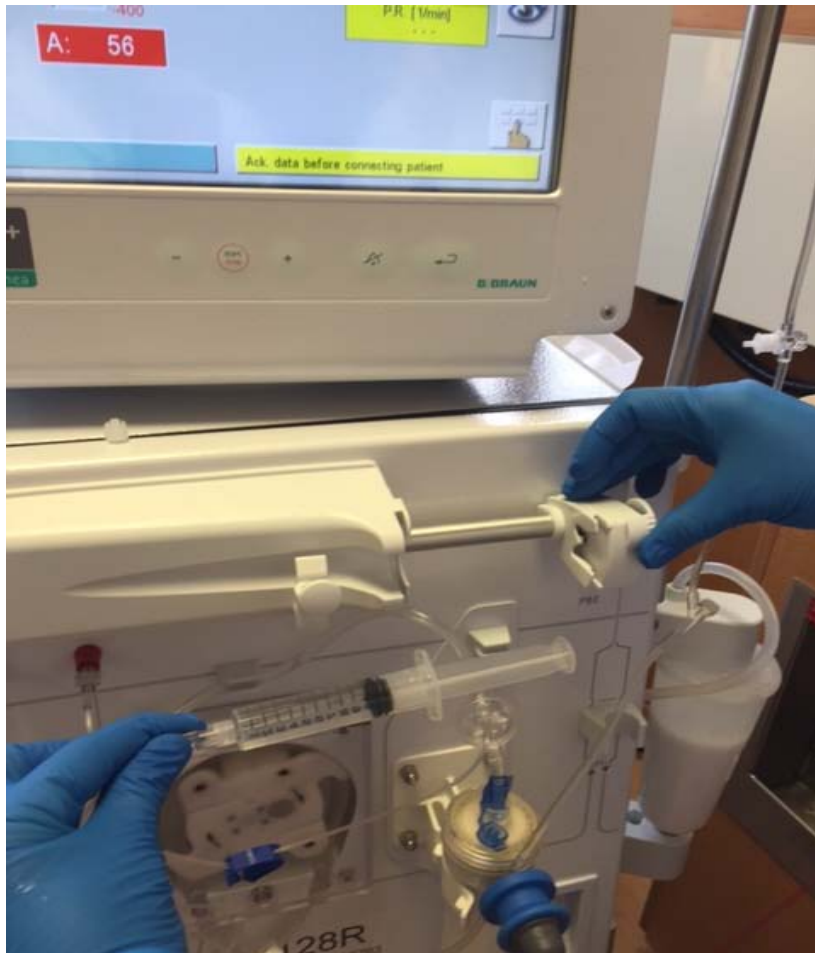

Release the unlocking lever by pushing tab up on the right.

Slide the clip to the right to open and place syringe.

Inserting the heparin syringe



Pull and turn syringe bracket down.

Insert syringe in such a way that the syringe wings are securely placed in the syringe gripping plate.


Turn syringe bracket up to hold syringe in place.

Inserting the heparin syringe

Slide clip toward syringe until aligned with syringe hub.

Pull syringe hub slightly forward with index finger, placing syringe hub in the clip.

Do not manually close the unlocking lever.

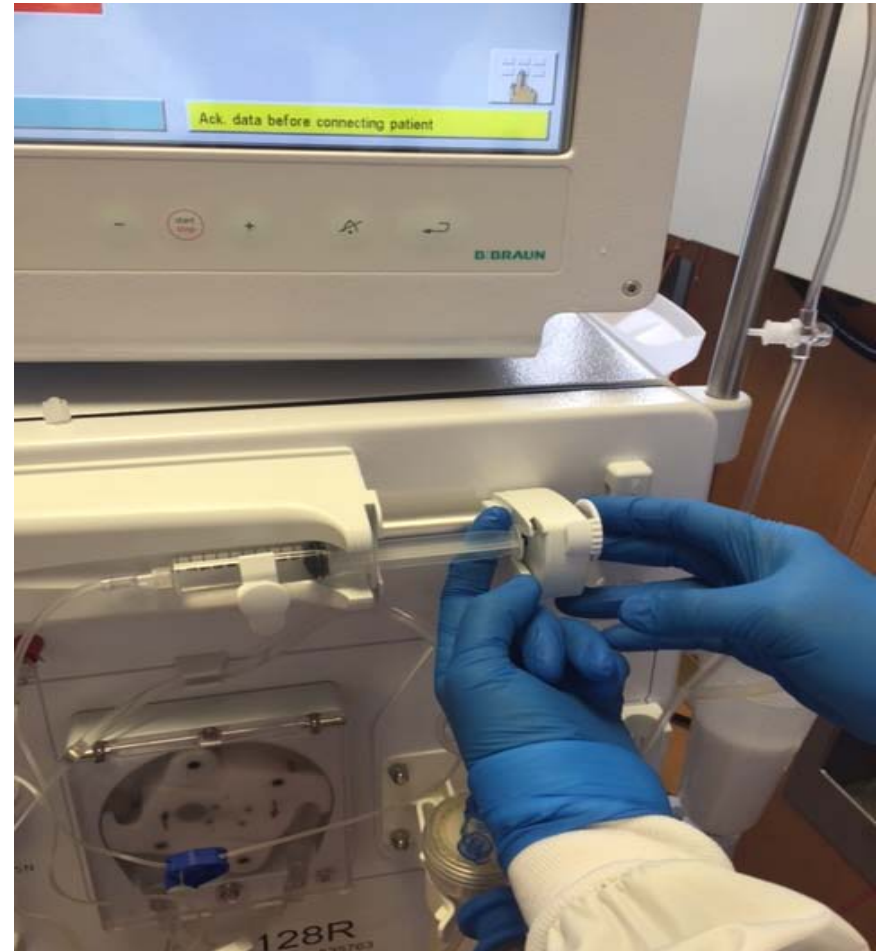

Inserting the heparin syringe

Push on syringe wings until unlocking lever automatically locks syringe into place.

If there is no sound (snap) when the lock engages, the syringe may not be properly loaded.



Inserting the heparin syringe

Unclamp heparin infusion clamp as per policy and procedure.

Set heparin parameters according to the patient prescription.



Thank you

For additional clinical resources please visit us online at:

www.BBraunUSA.com/Clinical