


Treatment Initiation using Streamline Bloodlines

Note: These step by step instructions are to be used as a guideline and do not take the place of Policy and Procedure. The information presented in this guide is not intended to replace the Instructions for Use for the Dialog⁺ hemodialysis system, the Instructions for Use for the Streamline bloodlines, and/or the Institution's policies and procedures. Before using the Dialog⁺ hemodialysis system and the Streamline bloodlines, users should read and be familiar with the complete Instructions for use and the institution's policies and procedures.



1. Press the start treatment icon . Verify patient treatment parameters and press the confirm button. Blood pump will stop automatically.

2. Double clamp saline line.

3. Unclamp heparin administration line if applicable.

4. Remove recirculation connector and attach venous and arterial blood lines to patients venous and arterial access.


5. Unclamp venous blood line and venous access.

6. Unclamp arterial bloodline and arterial access.

07. Press the "start/stop" button to start the blood pump. Pump speed will go to 200 automatically.

08. Increase the blood pump speed to prescribed rate by pressing the + button.

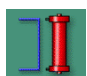

09. Press the  icon to take dialysate out of bypass and initiate treatment. The word "Hemodialysis" will appear at the top of the screen.



10. The Dialog will alarm and display "Patient connected?" in bottom left of screen. Press the mute button twice to mute and reset the alarm then press the "start/stop" button to restart the blood pump.




11. Press the  icon, then the  icon followed by the  icon to initiate the blood pressure cycler. (If not using patient card)

12. After the first set of vital signs the Dialog will display "ABPM check alarm limits" in the bottom right of



the screen. Press the  icon followed by the  icon. To set alarm limits on blood pressure and pulse press "Individual limit value adaptation" and then the confirm button or set each parameter limit individually by touching and changing the corresponding number.



13. Press the  icon twice to return to the main therapy screen.



B. Braun Medical Inc.
Renal Therapies Division
824 Twelfth Avenue
Bethlehem, PA 18018
Customer Service: 800-848-2066
24/7 Technical Service: 800-621-0445