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EN / AU Supplement to Dialog Instructions for Use (IFU): Information about the option ABPM and handling of old/new cuffs.

i Read the following information and instruct all users in your organization. Add this supplement to your Instructions for Use (IFU).

i This supplement only contains information that is changed. For the complete descriptions, refer to the existing Dialog Instructions for Use.

Symbols on ABPM Cuff

Symbol	Description
	Follow operating instructions in these instructions for use when operating the ABPM
	Type BF defibrillation proof applied part Classification acc. to IEC 60601-1
	Cuff is free of latex
	Upper arm diameter
	Cuff size: S (small), M (medium), L (large), XL(extra large) The respective size is indicated by the rectangle around the symbol.
	Marking for cuff placement

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Fig. 1 Male coupling on the machine

Fig. 2 Tubing female/female

- Check if your Dialog contains the male coupling (Fig. 1).
- Use tubing with two female couplings (Fig. 2).
- Connect tubing with one female coupling to machine.
- Connect same tubing with the other female coupling to cuff.
- For further measurement, follow your instructions for use Dialog.

Supplement ABPM Automatic Blood Pressure Measurement Module

Fig. 3 Female coupling on the machine

Fig. 4 Tubing female/male

- Check if your Dialog contains the female coupling (Fig. 3).
- Use tubing with one female coupling and one male coupling (Fig. 4).
- Connect tubing with male ending to machine.
- Connect tubing with female ending to cuff.
- For further measurement, follow your Instructions for Use Dialog.

Additional information for chapter ABPM / Cuff

Cuffs delivered by B. Braun are latex free. This is also indicated by the symbol on the cuff.

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Cuff sizes and accessories

Following cuff sizes and accessories are available:

Name	Size	Article Number (REF)
Small cuff (S)	Upper arm size 18 – 26 cm	7102372
Medium cuff (M)	Upper arm size 25 – 35 cm	7102771
Large cuff (L)	Upper arm size 33 – 47 cm	7102380
Extra large cuff (XL)	Upper arm size 42 – 54 cm	7102390
Tubing female/male	Length 3 m	7102698
Tubing female/female	Length 3 m	7102699

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Dialog® Advanced, Dialog+®, Dialog+® SW 9.xx Dialysis Machines

Supplement Option ABPM

CE 0123

CE marking according to directive 93/42/EEC
Technical alterations reserved

Rx only

B|BRAUN
SHARING EXPERTISE

Manufacturer:
B. Braun Avitum AG
34209 Melsungen, Germany
Tel +49 (56 61) 71-0
Fax +49 (56 61) 75-0

US Distributor:
B. Braun Medical Inc.
Avitum Division
Bethlehem, PA 18018-3524 USA
Made in Germany
Tel 1-800-621-0445
Fax (610) 691-1547

Supplement 38910463 / Rev. 2.87.30 / December 2018

Supplement ABPM Automatic Blood Pressure Measurement Module

YOU WILL NEED ONE COPY PER MACHINE. PLEASE COPY IF NECESSARY.

US Supplement to Dialog® Instructions for Use (IFU): Information about the option ABPM and handling of old/new cuffs.

- i** Read the following information and instruct all users in your organization. Add this supplement to your Instructions for Use (IFU).
- i** This supplement only contains information that is changed. For the complete descriptions, refer to the existing Dialog Instructions for Use.

Symbols on ABPM Cuff

Symbol	Description
	Follow operating instructions when operating the ABPM <i>IEC 60601-1 Medical electrical equipment – Part 1: General requirements for basic safety and essential performance; Reference number D.1 – 11</i>
	DEFIBRILLATION-PROOF TYPE BF APPLIED PART Classification acc. to IEC 60601-1 <i>IEC 60601-1 Medical electrical equipment – Part 1: General requirements for basic safety and essential performance; Reference number D.1 – 26</i>
	Cuff is not made with natural rubber latex <i>ISO 15223-1 Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements; Reference number 5.4.5; Annex B.2</i>
	Upper arm diameter
	Cuff size: S (small), M (medium), L (large), XL(extra large) The respective size is indicated by the rectangle around the symbol.
	Marking for cuff placement

Handling of old/new cuffs with the option ABPM

In order to improve therapy outcomes and patient comfort, B. Braun offers a new series of blood pressure cuffs for the option ABPM. To find out which module is assembled and cuff is needed, check the coupling in your machine and compare with the pictures below. Follow the appropriate instructions.

Fig. 1 Male coupling on the machine

Fig. 2 Tubing female/female

- Check if your Dialog contains the male coupling (Fig. 1).
- Use tubing with two female couplings (Fig. 2).
- Connect tubing with one female coupling to machine.
- Connect same tubing with the other female coupling to cuff.
- For further measurement, follow your Dialog instructions for use.

Supplement ABPM Automatic Blood Pressure Measurement Module

Fig. 3 Female coupling on the machine

Fig. 4 Tubing female/male

- Check if your Dialog® contains the female coupling (Fig. 3).
- Use tubing with one female coupling and one male coupling (Fig. 4).
- Connect tubing with male coupling to machine.
- Connect tubing with female coupling to cuff.
- For further measurement, follow your Dialog instructions for use.

Additional information for ABPM / Cuff

Cuffs delivered by B. Braun are not made with natural rubber latex. This is also indicated by the symbol on the cuff.

WARNING

Risk to the patient due to wrong measurements!

Using an unsuitable cuff will affect performance of the ABPM feature.

- Only cuffs delivered by B. Braun should be used. Other cuffs must be qualified for usage with the machine, e.g. by independent bodies.

Cuff sizes and accessories

The following cuff sizes and accessories are available:

Name	Size	Article Number (REF)
Small cuff (S)	Upper arm size 18 – 26 cm	7102372
Medium cuff (M)	Upper arm size 25 – 35 cm	7102771
Large cuff (L)	Upper arm size 33 – 47 cm	7102380
Extra large cuff (XL)	Upper arm size 42 – 54 cm	7102390
Tubing female/male	Length 3 m	7102698
Tubing female/female	Length 3 m	7102699