

Handling Instructions in Four Easy Steps



1 Open the bag's outer packaging on the side where the hanger holes are located and tear off the outer foil completely.



2 Unfold the bag and place it on a flat surface.



3 Squeeze the corners of the small chamber with both hands until the sealing seam between the two chambers starts to open.



4 Then, press on the large chamber with both hands to ensure the sealing seam is completely open. After mixing, the solution is ready for use.

NOTE: This informational sheet is not intended to replace the Instructions for Use for the B. Braun Bicarbonate Dialysate solution, or an institution's policies and procedures. Before using the B. Braun Bicarbonate Dialysate, Users should read and be familiar with the complete Instructions for Use and the institution's policies and procedures.

NOTE:

- Break inner cone above luer lock connection in a forward and backward direction to initiate flow.
- DO NOT USE if container seals or inner cone have been damaged.

B | BRAUN
SHARING EXPERTISE

Renal Therapies Division

Customer Service: 800-848-2066

24/7 Technical Service: 800-621-0445