

Solcart B®

Bicarbonate Cartridge for use with the Dialog+® Hemodialysis System

- Easy to Pierce
- Lightweight and Easy to Use
- Convenient to Store

Specifications:

Cartridge housing material: Polypropylene

Solcart B cartridge dry concentrate sizes:

- 650 grams sodium bicarbonate powder concentrate
- 760 grams sodium bicarbonate powder concentrate
- 1100 grams sodium bicarbonate powder concentrate

Piercing Point: Mesh filter for easy piercing



Reorder Information:

Item # 802	650g	10 units/carton
Item # 803	760g	10 units/carton
Item # 805	1100g	8 units/carton

Dialysate Flow Rate	Run Time with Solcart B* 650g	Run Time with Solcart B* 760g	Run Time with Solcart B* 1100g
500 (ml/min)	6.2 hrs	7.3 hrs	10.6 hrs
600 (ml/min)	5.2 hrs	6.1 hrs	8.8 hrs
700 (ml/min)	4.4 hrs	5.2 hrs	7.5 hrs
800 (ml/min)	4.1 hrs	4.8 hrs	6.9 hrs

*at bicarbonate setting of 35 mmol/L

Run time values based on testing conducted using 760g Solcart B Cartridge. Data extrapolated for other sizes and represent approximate run time values



For more information, please contact a
B. Braun Avitum Division representative
at 800-848-2066, or email rtd.us@bbraun.com