

# CARESITE Micro™ Extension Set

with CARESITE Micro Luer Access Device, Wedge™ Catheter Stabilizer, and Spin-Lock® Connector

**REF** 354278

**LENGTH** 8 in. (20 cm) (approx.)

**PRIMING VOLUME** 0.4 mL (approx.)



**STERILE EO**

Sterilized using ethylene oxide



Single Use



Do not re-sterilize



Non-pyrogenic



Single sterile barrier system

Rx only

Not made with natural rubber latex or DEHP.

A4809110-2 REV. 8/23

Intended for direct injection, intermittent infusion, continuous infusion, or aspiration.

**Instructions for Use:** Use Aseptic Technique.

- **To access CARESITE Micro Luer Access Device:** Swab top of luer access device vigorously for 15 seconds with 70% Isopropyl Alcohol and allow to air dry prior to use. Use a luer connector without a needle. After medication administration, flush per institutional protocol.
- **For use with Power Injectors (maximum pressure of 325 psi and maximum flow rate of 10 mL/sec):** Secure luer access device with a luer lock connector. Refer to labeling of associate devices (e.g. catheters) for maximum pressure and flow rate to ensure compatibility.

- Compatible with male luer connectors of internal diameter (ID) between 0.067 in. (1.7 mm) and 0.114 in. (2.9 mm).
- Replace set per CDC guidelines.
  1. Prime extension set to ensure air is expelled.
  2. Firmly attach connectors to desired connections.
  3. When using Wedge Catheter Stabilizer, with flat surface toward skin, slide Wedge Catheter Stabilizer on tubing and firmly position onto patient connector.
  4. Secure Wedge Catheter Stabilizer to patient with tape, applying firm but gentle pressure over all surfaces of Wedge Catheter Stabilizer.

**B. Braun Medical Inc.**

Bethlehem, PA 18018-3524 USA  
1-800-227-2862

**B | BRAUN**