






CARESITE Micro™ Smallbore Extension Set

with CARESITE Micro Luer Access Device and Spin-Lock® Connector

REF	354265		STERILE EO	Sterilized using ethylene oxide	 Single Use	 Do not resterilize
LENGTH	8 in. (20 cm) (approx.)		 Non-pyrogenic	 Single sterile barrier system	Rx only	
PRIMING VOLUME	0.3 mL (approx.)		Not made with natural rubber latex or DEHP.		A4809112-2 REV. 8/23	

Intended for direct injection, intermittent infusion, continuous infusion, or aspiration.

Instructions for Use: Use Aseptic Technique.

- **To access CARESITE Micro Luer Access Device:** Swab top of luer access device vigorously for 15 seconds with 70% Isopropyl Alcohol and allow to air dry prior to use. Use a luer connector without a needle. After medication administration, flush per institutional protocol.
- Compatible with male luer connectors of internal diameter (ID) between 0.067 in. (1.7 mm) and 0.114 in. (2.9 mm).

- **For use with Power Injectors (maximum pressure of 325 psi and maximum flow rate of 10 mL/sec):** Secure luer access device with a luer lock connector. Refer to labeling of associate devices (e.g. catheters) for maximum pressure and flow rate to ensure compatibility.
- Replace set per CDC guidelines.
 1. Prime extension set to ensure air is expelled.
 2. Firmly attach connectors to desired connections.

B. Braun Medical Inc.

Bethlehem, PA 18018-3524 USA
1-800-227-2862

B | BRAUN