






CARESITE Micro™ Extension Set

with CARESITE Micro Luer Access Device and Spin-Lock® Connector

| | | | | | | |
|-----------------------|-------------------------|---|---|---|--|--|
| REF | 354267 |  | STERILE EO | Sterilized using ethylene oxide |  Single Use |  Do not resterilize |
| LENGTH | 8 in. (20 cm) (approx.) | |  Non-pyrogenic |  Single sterile barrier system | Rx only | |
| PRIMING VOLUME | 0.4 mL (approx.) | | Not made with natural rubber latex or DEHP. | | | A4809113-2 REV. 8/23 |

Intended for direct injection, intermittent infusion, continuous infusion, or aspiration.

Instructions for Use: Use Aseptic Technique.

- **To access CARESITE Micro Luer Access Device:** Swab top of luer access device vigorously for 15 seconds with 70% Isopropyl Alcohol and allow to air dry prior to use. Use a luer connector without a needle. After medication administration, flush per institutional protocol.
- Compatible with male luer connectors of internal diameter (ID) between 0.067 in. (1.7 mm) and 0.114 in. (2.9 mm).

- **For use with Power Injectors (maximum pressure of 325 psi and maximum flow rate of 10 mL/sec):** Secure luer access device with a luer lock connector. Refer to labeling of associate devices (e.g. catheters) for maximum pressure and flow rate to ensure compatibility.
- Replace set per CDC guidelines.
 1. Prime extension set to ensure air is expelled.
 2. Firmly attach connectors to desired connections.

B. Braun Medical Inc.

Bethlehem, PA 18018-3524 USA
1-800-227-2862

B | BRAUN