

CARESITE Micro™ Smallbore Extension Set

with removable CARESITE Micro Luer Access Device and Spin-Lock® Connector

REF 354269

LENGTH 8 in. (20 cm) (approx.)

PRIMING VOLUME 0.3 mL (approx.)



STERILE EO Sterilized using ethylene oxide

 Single Use

 Do not resterilize

 Non-pyrogenic  Single sterile barrier system

 CAUTION

Not made with natural rubber latex or DEHP. Rx only A4809115-2 REV. 8/23

 **CAUTION:** SET CONTAINS REMOVABLE LUER ACCESS DEVICE.

Intended for direct injection, intermittent infusion, continuous infusion, or aspiration.

Instructions for Use: Use Aseptic Technique.

- Tighten luer access device before use.
- **To access CARESITE Micro Luer Access Device:** Swab top of luer access device vigorously for 15 seconds with 70% Isopropyl Alcohol and allow to air dry prior to use. Use a luer connector without a needle. After medication administration, flush per institutional protocol.
- Compatible with male luer connectors of internal diameter (ID) between 0.067 in. (1.7 mm) and 0.114 in. (2.9 mm).

- **For use with Power Injectors (maximum pressure of 325 psi and maximum flow rate of 10 mL/sec):** Secure luer access device with a luer lock connector. Refer to labeling of associate devices (e.g. catheters) for maximum pressure and flow rate to ensure compatibility.
- Replace set per CDC guidelines.
 1. Prime extension set to ensure air is expelled.
 2. Firmly attach connectors to desired connections.

B. Braun Medical Inc.
Bethlehem, PA 18018-3524 USA
1-800-227-2862

B | BRAUN