

Antimicrobial Effectiveness Testing

Successfully meets USP<51> Category 2 Criteria

Preservative Testing

No microbial growth was observed at 7, 14 and 28 days for the following organisms:

Acinetobacter baumannii

Enterobacter cloacae

Enterococcus faecalis

Vancomycin Resistant *Enterococcus faecalis* (VRE)

Escherichia coli

Proteus mirabilis

Pseudomonas aeruginosa

Serratia marcescans

Staphylococcus aureus

Methicillin Resistant *Staphylococcus aureus* (MRSA)

Staphylococcus epidermidis

Candida albicans

Preservative testing demonstrated using USP<51>, with an expanded list of organisms and additional 7 day time-point, to support the role of PHMB as a preservative to inhibit microbial growth within the product during shelf storage.

Prontosan® Wound Irrigation Solution **contains:**

0.1 % Undecylenamidopropyl Betaine, 0.1 % Polyaminopropyl Biguanide (Polyhexanide [PHMB]), Purified Water

pH may be adjusted with Sodium Hydroxide

Application: For acute wounds rinsing may be sufficient. For wounds covering a large area and wounds difficult to access, bathe the whole section of the body with Prontosan Wound Irrigation Solution, for at least 15 minutes. The solution should be used in undiluted form. Dressings or scabs sticking to wounds can be removed by soaking with Prontosan solution.

Warning: Do not infuse, inject, or ingest. For external use only.

Store at room temperature. Avoid excessive heat. Do not refrigerate or freeze.

Use this product within 28 days after opening.

Date Opened:

Close bottle immediately after use.

880260
P-7247-6
REV. 1/20

Prontosan® Wound Irrigation Solution

For cleansing wounds. Also may be used for moistening and lubricating wound dressings.



LOT

REF 400441

GTIN: 04046964295775

STERILE A

Aseptic filtration <1 CFU/mL, until the product is first opened.

NONPYROGENIC fluid and fluid path prior to removal of cap.



For single patient use only. Do not resterilize.

Rx only



See Instructions for Use.

Not made with natural rubber latex or DEHP.

Manufactured for:
B. Braun Medical Inc.
Bethlehem, PA 18018-3524 USA
1-800-227-2862
Made in Germany.

350 mL

B | BRAUN