

STEADYCare™ Extension Set

with Wedge™ Catheter Stabilizer, CARESITE® Luer Access Device, and Spin-Lock® Connector

REF 470188

LENGTH 8.5 in. (22 cm) (approx.)

PRIMING VOLUME 0.7 mL (approx.)

8.5
in.

STERILE EO Sterilized using ethylene oxide

2 Single Use

2 Do not resterilize

~~XX~~ Non-pyrogenic


○ Single sterile barrier system

Rx only

Not made with natural rubber latex or DEHP.

Pat.: www.bbraunusa.com/patents

A4805351-2 REV. 11/23

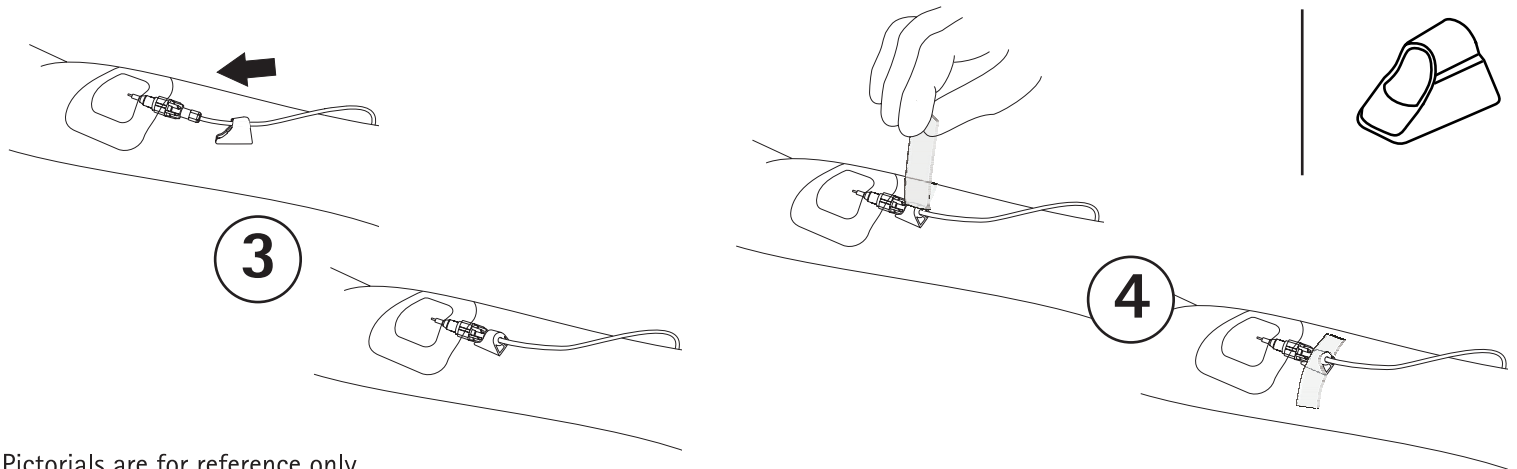

Pictorials are for reference only.

Instructions for Use: Use Aseptic Technique.

- **For use with Power Injectors** (maximum 300 psi and maximum flow rate 10 mL/sec.): Refer to labeling of associate devices for maximum pressure and maximum flow rate to ensure compatibility.
 - **To access CARESITE Luer Access Device:** Swab top of luer access device vigorously for 15 seconds with 70% Isopropyl Alcohol and air dry prior to use. Use a luer connector without a needle. After medication administration, flush per institutional protocol, attaching completely between flushes.
 - Replace set per CDC guidelines.
1. Prime extension set to ensure air is expelled.
 2. Firmly attach connectors to desired connections.
 3. When using Wedge Catheter Stabilizer, with flat surface toward skin, slide Wedge Catheter Stabilizer on tubing and firmly position onto patient connector.
 4. Secure Wedge Catheter Stabilizer to patient with tape, applying firm but gentle pressure over all surfaces of Wedge Catheter Stabilizer.

B. Braun Medical Inc.
Bethlehem, PA 18018-3524 USA
1-800-227-2862

B | BRAUN