


OnGuard® 2 Closed System Transfer Device Catheter Adaptor

Reconstitution and Administration Devices

B|BRAUN
SHARING EXPERTISE

Technical Datasheet

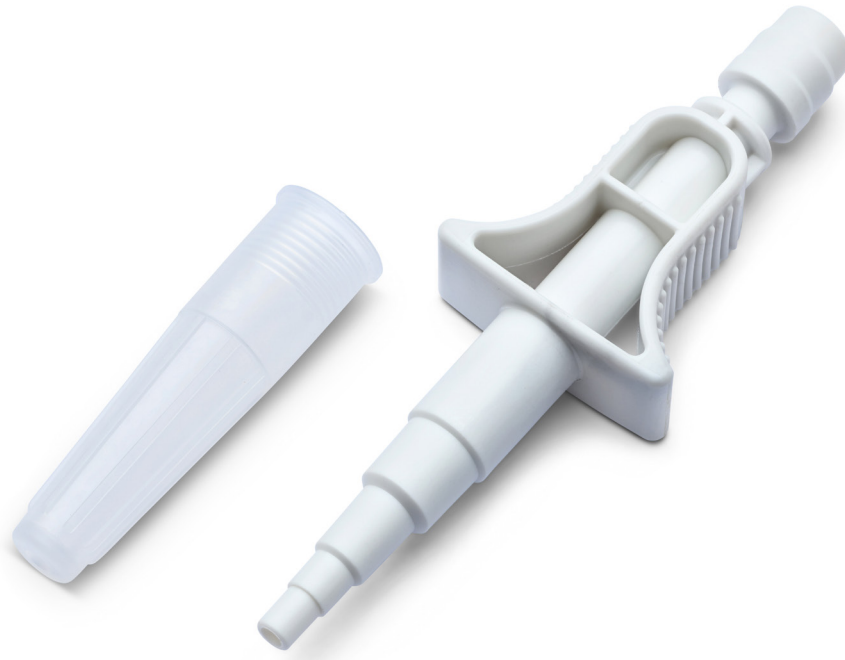

- The OnGuard 2 Catheter Adaptor is intended for the safe administration of drugs directly into a urinary catheter.
- The OnGuard 2 Catheter Adaptor has an ergonomic design that fits standard urinary catheters and converts an open catheter connection into a closed OnGuard 2 Connection.
- An OnGuard 2 Syringe Adaptor/Syringe Adaptor Lock and OnGuard 2 Vial Adaptor are used to safely transfer the drug from the vial to the syringe. The syringe adaptor is then connected for administration to the patient's catheter via the OnGuard 2 Catheter Adaptor.
- A "click" sound indicates a safe, closed connection between the two OnGuard 2 Devices.
- A simple push of the syringe adaptor/syringe adaptor lock wings will safely disconnect the syringe adaptor lock from the OnGuard 2 Catheter Adaptor.
- The OnGuard 2 Catheter Adaptor is not made with DEHP, PVC or natural rubber latex.
- The OnGuard 2 Catheter Adaptor is compatible with disinfection products/procedures such as alcohol (e.g. EtOH, IPA) and sporicidal-based disinfectants.¹

Intended use on IFU:²

The OnGuard 2 Catheter Adaptor is a single-use, sterile Closed System Transfer Device (CSTD) that mechanically prohibits the release of drugs, including antineoplastic and hazardous drugs, in vapor, aerosol or liquid form during administration, thus minimizing exposure of individuals, healthcare personnel and the environment to hazardous drugs. The OnGuard 2 Catheter Adaptor prevents the introduction of microbial and airborne contaminants into the drug or fluid path for up to seven days and 10 activations.³

OnGuard®2 Closed System Transfer Device Catheter Adaptor

Specifications

Order information

Product Code	412182
Ordering Unit	Case
Units per case	50/box, 100/case
WSN number	08021-4121-82

Product technical data

External fitting	Urinary catheter
Flow path volume	0.7 mL
Septum piercings	Up to 10 times
Shelf life	3 years
Sterilization	Ethylene oxide

Material specifications

Component	Material
Connector for urinary catheter	Polypropylene
Protective vented cap	Polyethylene
Elastomeric sealing	Polyisoprene

FDA 510(k) cleared under the ONB product code

Disposal of components should be according to local regulations and procedures for handling of hazardous drugs

References:

- https://www.simplivia.com/files/pdf/Tevadaptor_Specs/Tevadaptor_spec_catheter_adaptor_0518.pdf
- OnGuard 2 Instructions for Use
- Chemfort™ Resource Center | Simplivia. www.simplivia.com. Accessed October 13, 2023. https://www.simplivia.com/files/pdf/Chemfort_Brochures/Simplivia_Chemfort_10_Activations.pdf

B. Braun Medical Inc. | Bethlehem PA | 800-227-2862 | [bbraunusa.com](https://www.bbraunusa.com)

OnGuard 2 CSTD is manufactured by Simplivia Healthcare Ltd. and distributed by B. Braun Medical Inc.
Rx only. ©2024 B. Braun Medical Inc., Bethlehem, PA. All rights reserved IVX101_5/24