

# Prontosan® Wound Irrigation Solution

For cleansing wounds. Also may be used for moistening and lubricating wound dressings.



LOT

REF 400443

GTIN: 04046964295836

Data  
Matrix  
Barcode

## Antimicrobial Effectiveness Testing

Successfully meets USP<51> Category 2 Criteria

## Preservative Testing

No microbial growth was observed at 7, 14 and 28 days for the following organisms:

*Acinetobacter baumannii*  
*Enterobacter cloacae*  
*Enterococcus faecalis*  
Vancomycin Resistant *Enterococcus faecalis* (VRE)  
*Escherichia coli*  
*Proteus mirabilis*  
*Pseudomonas aeruginosa*  
*Serratia marcescens*  
*Staphylococcus aureus*  
Methicillin Resistant *Staphylococcus aureus* (MRSA)  
*Staphylococcus epidermidis*  
*Candida albicans*

Preservative testing demonstrated using USP<51>, with an expanded list of organisms and additional 7 day time-point, to support the role of PHMB as a preservative to inhibit microbial growth within the product during shelf storage.

Manufactured for:  
**B. Braun Medical Inc.**  
Bethlehem, PA 18018-3524 USA  
1-800-227-2862  
Made in Germany.

880367  
P-8041-5  
REV. 1/20

1000 mL

**B | BRAUN**



For single patient use only.  
Do not resterilize.

STERILE | A

Not made with natural rubber latex or DEHP.

Aseptic filtration  
<1 CFU/mL, until the  
product is first opened.

NONPYROGENIC fluid  
and fluid path prior to  
removal of cap.



See Instructions  
for Use.

Rx only

## Prontosan® Wound Irrigation Solution contains:

0.1% Undecylenamidopropyl Betaine,  
0.1% Polyaminopropyl Biguanide  
(Polyhexanide [PH B]), Purified Water  
pH may be adjusted with Sodium Hydroxide

**Application:** For acute wounds rinsing may be sufficient. For wounds covering a large area and wounds difficult to access, bathe the whole section of the body with Prontosan Wound Irrigation Solution, for at least 15 minutes. The solution should be in undiluted form. Dressings or scabs sticking to wounds can be removed by soaking with solution.

**Warning:** Do not infuse, inject, or ingest. For external use only.

Store at room temperature.  
Avoid excessive heat.  
Do not refrigerate or freeze.

Use this product within 28 days after opening. Close bottle immediately after use.

Date Opened: